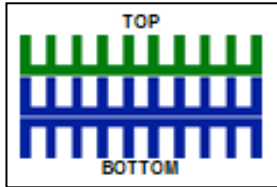
Geo-Matrix Gold & Gold+

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38 Lincoln St., Hamilton, ON L8L 7L5
www.blakemedical.ca

[Cleaning your Geo-Matrix cushion]

For daily cleanings all that is needed is a wipe down of the outer cover. This outer cover is fluid proof fabric and will prevent most fluids from permeating into the Geo-Matrix gel inside. For these daily cleanings soap and water will be all that is needed. If fluids do soil the inner layers of the Geo-Matrix cushion follow the instructions below.

The cleaning properties of the Geo-Matrix cushion are unique; the covers are machine washable and can go in the dryer on low heat. To clean, disassemble the cushion into the component parts and wash the cover on a delicate cycle. Avoid drying in direct sunlight.

The Geo-Matrix gel cannot go into the washing machine as it is too heavy and could damage your machine. To clean the Geo-Matrix gel, spray it with a mild non-abrasive detergent and wipe clean. In-house, Blake Medical uses "Urine Off", "Virox" and "Sanicare" to disinfect Geo-Matrix product. Feel free to contact us with any questions about a disinfectant you may have prior to its use.

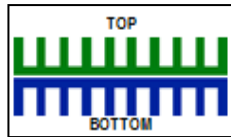
Please note: do not use bleach to clean any Geo-Matrix product. Improperly diluted bleach will eventually corrode the Geo-Matrix gel.

[Reassembling your Geo-Matrix cushion after cleaning]

Each cushion cover contains a patch indicating the correct placement and orientation of the gel layers. The Geo-Matrix Bronze, Silver and Gold/Gold+ cushions are reassembled as follows:


Geo-Matrix Bronze

This two-layer cushion is assembled with the large column blue layer of gel on the bottom with the open end of the columns facing down. The second layer (green) is placed on top of the blue layer with the open end of the columns facing up. The layers are then covered with the inner incontinent cover and inserted into the outer cover.


Geo-Matrix Silver

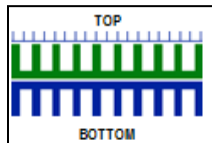This three-layer cushion is assembled with the large column blue layer of gel on the bottom with the open end of the columns facing down. The second large layer of the cushion, a green layer, is placed in top of the bottom blue layer with the open end of the columns facing up. The third layer, the pillow-top layer, is then placed on top of the green layer with the columns facing up. The layers are then covered with the inner incontinent cover and inserted into the outer cover.

Geo-Matrix Gold & Gold+

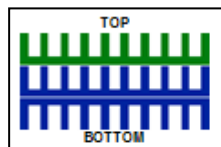This three-layer cushion is assembled with the large column blue layer of gel on the bottom with the open end of the columns facing down. The second large layer of the cushion, another blue layer, is placed in top of the bottom blue layer with the open end of the columns facing up. The third layer, either blue or green, is then placed on top of the green layer with the columns facing up. The layers are then covered with the inner incontinent cover and inserted into the outer cover.

[Care of your Geo-Matrix cushion]

The cushion is designed to be used at room temperature. If it is subjected to extreme heat or cold, allow the cushion to acclimate to room temperature before using.

The gel layers are light sensitive and will begin to breakdown if exposed for prolonged periods to direct sunlight. Avoid air-drying outside and exposure of the gel layers to direct sunlight.

The gel layers are designed to be covered with a inner and outer cover. To ensure that the cushion retains its fire-retardant characteristics, ensure that during reassembly both covers are used.

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